# Revamil®

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#### The Product

Revamil® Balm

15 g ointment for the treatment of superficial wounds

#### Ingredients

Arachis oleum, cera alba, glyceryl oleate, mel, aqua

#### Product description

Revamil® Balm is a nourishing ointment for the treatment of superficial wounds. Revamil® Balm consists of a base ointment in combination with pure honey. The honey in Revamil® Balm contains high levels of honey enzymes, like glucose oxidase. In a moist environment, this enzyme produces low levels of hydrogen peroxide. The high sugar concentration of honey facilitates mixing of wound moisture with Revamil® Balm, thus forming an intermediate layer between the wound and the honey, which gives anti-bacterial protection. Revamil® Balm does not contain residues from pesticides, herbicides, conservatives or other components that are known to cause allergic reactions.

#### Instructions for use

- Apply a thin layer of Revamil® Balm on the wound
- The wound can be covered with a wound dressing
- Treat affected areas 1 to 2 times per day

#### Indications

Revamil® Balm can be applied on superficial wounds. Revamil® Balm is allowed for all age groups

#### Contra indications

Do not use Revamil® Balm if the patient is known to be allergic to honey.

#### Side effects

- During the first minutes after application on minor wounds Revamil® Balm can cause a light stinging sensation
- In case of unexpected side effects after use contact your doctor or the manufacturer

#### Safety precautions

- Do not use Revamil® Balm if the tube is damaged
- Always replace the top on the tube after use
- Do not allow the tube to come into direct contact with the surface of the wound
- Once the tube has been opened use the contents within 3 months
- Do not treat different patients with the same tube

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Bfactory Health Products B.V., Remmerden 58, 3911 TZ Rhenen (NL)