Revamil®

- Spread a thin layer of Revamil on the secondary wound dressing.
- Revamil can also be applied directly to the wound and then covered with a secondary dressing.
- Attach the secondary dressing in the normal way.
- Depending on the nature of the wound, replace the dressing every day or every two days.

Indications
Revamil wound gel can be used to treat chronic sores (bed sores and different kinds of ulcers), infected wounds, acute wounds, surgical oncological wounds, radiation induced oncological wounds and minor (first- and second-degree) burns.

Contra indications

- Do not use Revamil if the patient is known to be allergic or hypersensitive to honey.

Side effects

- In rare cases Revamil wound gel can cause a momentary stinging sensation.
- In case of unexpected side effects after use contact your doctor or the manufacturer.

Safety precautions

- Do not use the gel if the tube is damaged.
- Always replace the top on the tube after use.
- Before applying Revamil, wet the wound with 0.9% NaCl.
- Do not allow the tube to come into direct contact with the surface of the wound.
- Use the contents within 3 months after opening tube.
- Do not treat different patients with the same tube.
- Do not treat a wound with this product for more than 30 days without interruption.

Storing conditions
Store between 10 and 30°C

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