XTRATA[®] VERBANDFOLIE

wasserdicht
waterproof

atmungsaktiv
breathable

transparent
transparent

VORWORT *FOREWORD*

DR. GERLINDE LUCH

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The LKH (hospital) of Steyr is a key aspect hospital with two headquarters - hospital Steyr and center Enns - it belongs to OÖ Gesundheits- und Spitals AG (Gespag).

15 departments and 6 institutes take care for more than 41.000 residential patients from the district, as well as conducting 70.000 out-patient assessments. 2000 members of staff provide the best possible treatment at the Steyr hospital as well as at the center at Enns, the hospital is a significant local employer.

The wound management department of LKH Steyr was established over 10 years ago. The close collaboration between the doctors and wound managers is vital and resulted in the treatment of 3000 wound care patients in 2011. The wound management team is quite open-minded regarding innovative wound care products as well as treatment options. XTRATA® has been utilised for the past 3 years as an example. The patients appreciate the excellent next-to-skin comfort and the ease of application..

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XTRATA® - waterproof breathable and transparent combined film with STRATAGEL™-technology

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XTRATA® transparent films permit excellent moisture transfer, are combined with the new and unique STRATAGEL™ Technology. This results in less damage to the stratum corneum and the stated gentleness to the skin.

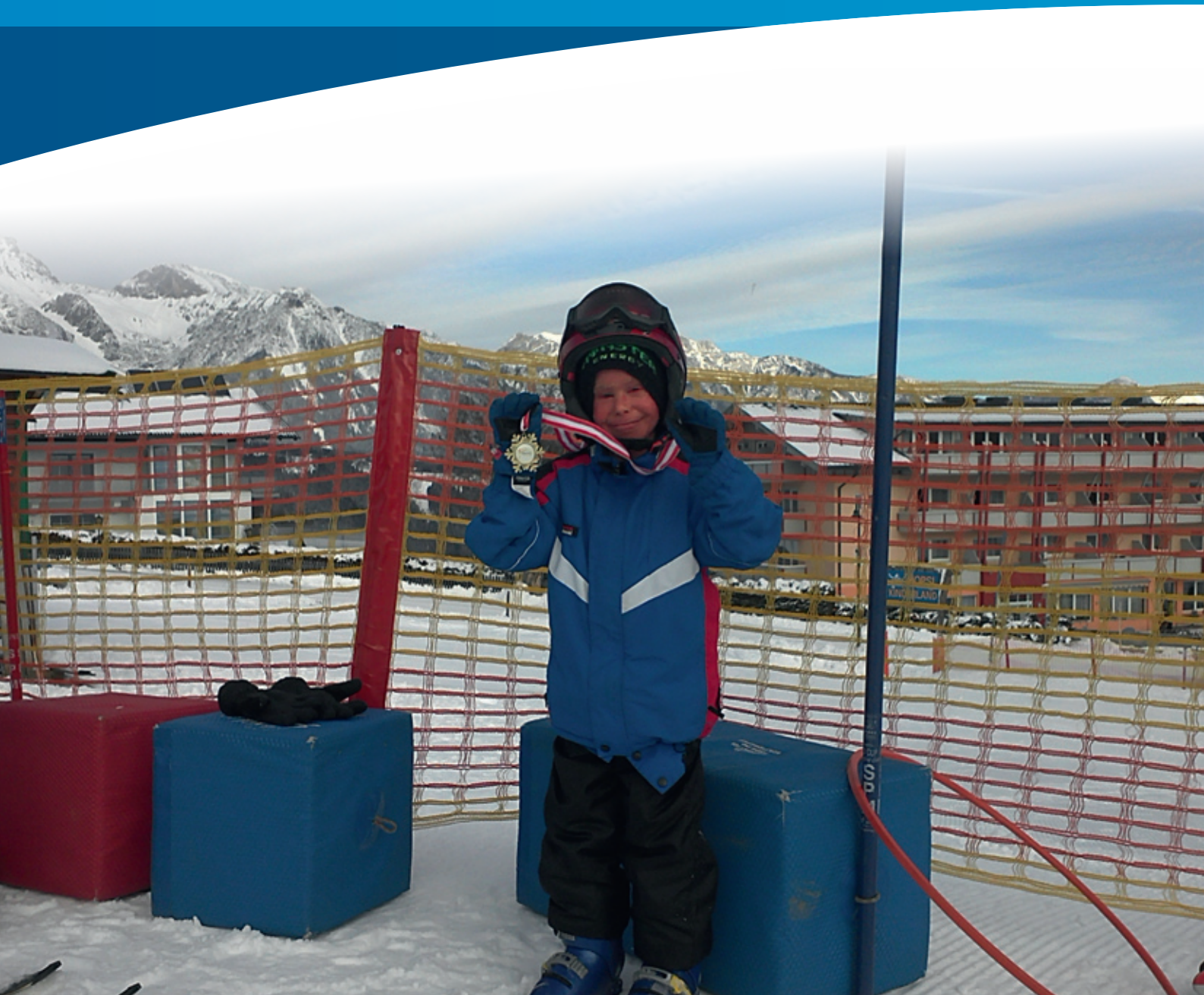
Advantages:

1. It acts as a barrier against germs from outside
2. It has a high and excellent Moisture Vapour Transfer Rate (MVTR).
3. Perfect adhesion with only a slight dissolution of the Stratum Corneum
4. Excellent ability to adapt and flexibility when applied on joints
5. Waterproof
6. Transparent surface allows observation of the applied spot
7. Latex Free
8. Easy application and painless removal

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This is Raphael

Raphael is 10 years old. Last december he learned skiing in Schladming. Raphael suffers from a very rare skin scale disease. In order to maintain the mobility, the scales have to be moved daily. In the past this happened in a painful and long procedure - today Raphael has got a basin with nibble fish. These fish allow him the gentle ablation of his skin scales.

Every morning before school at 5 o'clock he places himself in the basin with sea water and is treated by his fish. The nibbling of these fish is not displeasing. It's just disagreeable if Raphael has got small injuries because sea water burns in wounds. As needed the little wounds are protected with an XTRATA® film and Raphael is able to enjoy his tiny therapists.

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This is Mr Bauer

Mr. Bauer is 60 years old. He suffers from insulin-dependent diabetes mellitus. In November 2010 he had almost lost his left leg. Although he broke his left hand one year later which had to be operated upon, Mr. Bauer has a good reason for smiling.

In November 2010, his 4th toe had to be amputated. Due to a preceding gangrene condition the amputation of his 3rd and 5th toes was also necessary. In spite of everything Mr. Bauer was in luck, he kept his leg and now lives an ordinary life.

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Mr. Bauer has a very sensitive skin.
To protect the skin XTRATA® was applied
before negative pressure system.

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This is Florentina

...on her 18th birthday. When exercising, Florentina uses tape bandages. The tape adhering to the skin supports the capsule-ligament apparatus of the joint in question and can avoid joint swelling. In order to wear the colored bandages longer, they are protected by a XTRATA® film. So nothing keeps her from jumping into the swimming pond or taking a refreshing shower.

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EUROPA *EUROPE*

Focusmed

Innovation in wound care

Wäschergasse 15
A-5020 Salzburg
AUSTRIA

Agentur Scherrer GmbH

Doris Scherrer
Founder and Managing Director

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DISTRIBUTION PARTNER

Mediset Clinical Products GmbH

Ulf Högberg
Founder and Managing Director

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Precautions

- Do not apply directly on wounds
- Do not reuse
- If skin damage occurs, remove the dressing and provide appropriate treatment
- If infection occurs, remove the dressing and provide appropriate treatment
- If necessary, disinfect the skin prior to use
- The skin should be dry prior to application to ensure adequate adhesion
- Mechanical skin damage may result if tension is applied when fixing the tape to the skin
- After use, the dressing should be disposed as medical waste

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