ES

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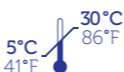
#### Antimicrobial, conductive and odour adsorbing primary dressing.

**STERILE R** Gamma irradiation sterilised. Do not use if pack is open or damaged. For single use only. Store in a dry place (>5 <30 °C/>41 <86 °F). Avoid exposure to temperatures above 50 °C/122 °F.

#### Description

Zorflex® dressing consists of a low adherent 100% pure activated carbon cloth. Zorflex® is a highly conformable dressing that follows the body contours to maintain contact with the wound surface. Zorflex® 100% pure activated carbon cloth provides an effective anti-microbial barrier to microbial contamination protecting the wound from invasive micro-organisms which may prevent the wound from healing. Zorflex® 100% pure activated carbon cloth is also effective against micro-organisms already present in the wound. The anti-microbial barrier properties of Zorflex® remain effective for a minimum of 7 days. Zorflex® has been shown to reduce odour, pain and exudation. Zorflex® is also conductive. Zorflex® can be used either dry or moistened with sterilised water.

#### Indications

Zorflex® is indicated as an antimicrobial dressing over dry or discharging, partial and full thickness wounds such as: cavities and fistulas, trauma, surgical sites, fungating carcinomas, pressure ulcers, venous ulcers, diabetic ulcers, burns, donor and recipient graft sites.

#### Contra-indications

None.

#### Precautions

Use sterile water to cleanse, irrigate or moisten the wound. The use of oil based products, topical ointments or saline will reduce the effectiveness of Zorflex®.

#### Instructions for use

- Cleanse the wound with sterile water.
- Zorflex® will tear easily. To avoid fraying cut the dressing to shape, covering the periwound if maceration is evident.
- Without stretching the dressing, apply Zorflex® either way up to the wound surface.
- The dressing should be applied with the direction of stretch running along the limb to allow movement.
- Secure the Zorflex® dressing in place using an appropriate secondary dressing or compression therapy. Alternatively secure with a dressing retention sheet, tape or bandage. In the case of highly exuding wounds an absorbent secondary dressing is appropriate. Zorflex® allows transfer of the exudate into the secondary dressing which should be replaced as necessary without disturbing the Zorflex® or the wound bed.
- Should the dressing dry or adhere to the wound, moisten or soak the dressing with sterile water to assist removal and avoid disruption of the healing wound.
- Zorflex® can be left on the wound for up to 7 days, or as clinically indicated. Any threads or fibres which detach from the dressing can be easily seen and removed.
- Device should only be used as specified.
- Device should be used under relevant supervision by healthcare professionals.

#### Product Availability

Product Availability Code	Size	Availability
ZF05	5cm x 5cm (2" x 2")	Pouched only
ZF10	10cm x 10cm (4" x 4")	Pouched only
ZF15	15cm x 25cm (6" x 10")	Pouched only
ZF20	20cm x 10cm (8" x 4")	Pouched only

#### Manufacturer

Chemviron Carbon Cloth Division, Rainton Bridge Industrial Estate  
Houghton-le-Spring, Tyne & Wear, DH4 5PP, GB

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Zorflex® trade mark of Chemviron Carbon. ©Chemviron Carbon.